

Support Plus™

FOLDING SHOWER SEAT

Thank You for your purchase from Support Plus™.

Please read all instructions thoroughly before use. Keep instructions for reference.



CAUTION:

- DO NOT EXCEED WEIGHT LIMIT OF 220 LBS.
 - DO NOT use on slippery, wet, or uneven surfaces
 - DO NOT use if damaged, or worn.
 - DO NOT expose to fire, weather, or chemical corrosion.
 - DO NOT attempt to repair damaged or missing parts.
 - DO NOT use seat for anything other than its intended use.
 - DO NOT stand or kneel on seat.
 - DO NOT attempt to move seat while in use.
 - DO NOT use outdoors.
- NOTE: Product is not designed to be immersed in water.

SAFETY GUIDELINES:

- When using seat, basic safety precautions should always be followed.
- Ensure that seat lock is fully engaged before use.
- Inspect all parts prior to each use to insure that all parts are securely fastened and in good working condition.
- Keep seat dry and clean, free from grease, oil, mud, snow, ice, and other slippery materials.
- Always keep body centered on seat while in use.
- Seat is meant for one person only.
- Keep dry while not in use.
- Seat may become slippery when wet. Please use caution while using in a bathtub or shower.
- To fold for storage, disengage locking pin located underneath the left side of the seat, then lift up on back of seat.

Please visit our web site:
www.SupportPlus.com
1-866-553-8875